


Confirmatory study of the IO Score, a tumor immune microenvironment (TIME) classifier, demonstrates efficacy in a real-world cohort of metastatic urothelial cancer (mUC) patients treated with immune checkpoint inhibitors (ICIs)

Tyler J. Nielsen, Robert S. Seitz, Matthew G. Varga, Catherine T. Cronister, David R. Hout, Brock L. Schweitzer, Douglas T. Ross, Kim McGregor
Oncocyte Corporation, Nashville, TN USA

Abstract# 1272


Background

- Prognosis for patients with mUC is poor with fewer than 10% surviving two years after diagnosis.
- Approximately 17% of patients receive a subsequent therapy, emphasizing the importance of therapeutic sequencing, despite multiple approved targeted therapies¹.
- Tumor mutational burden (TMB) has been shown to be an effective biomarker for predicting ICI therapy, but relatively few patients are TMB high.
- The 27-gene IO assay (IO Score) is a gene expression panel which has recently been described as a classifier of the tumor immune microenvironment (TIME) and has demonstrated ability to predict response to immune checkpoint inhibitors (ICIs) across multiple indications.^{2,3} The IO score produces a continuous result, from which a threshold of positivity has been prospectively set and remains consistent across NSCLC, TNBC, and UC.
- A more accurate predictor of response to ICIs in UC could better help direct patients to appropriate therapy earlier in their treatment.
- Therefore, we sought to expand the IMvigor210 findings of the utility of the IO Score as a biomarker for ICI efficacy in bladder cancer and suggest a new biomarker paradigm to aid in therapeutic sequencing for mUC.⁴

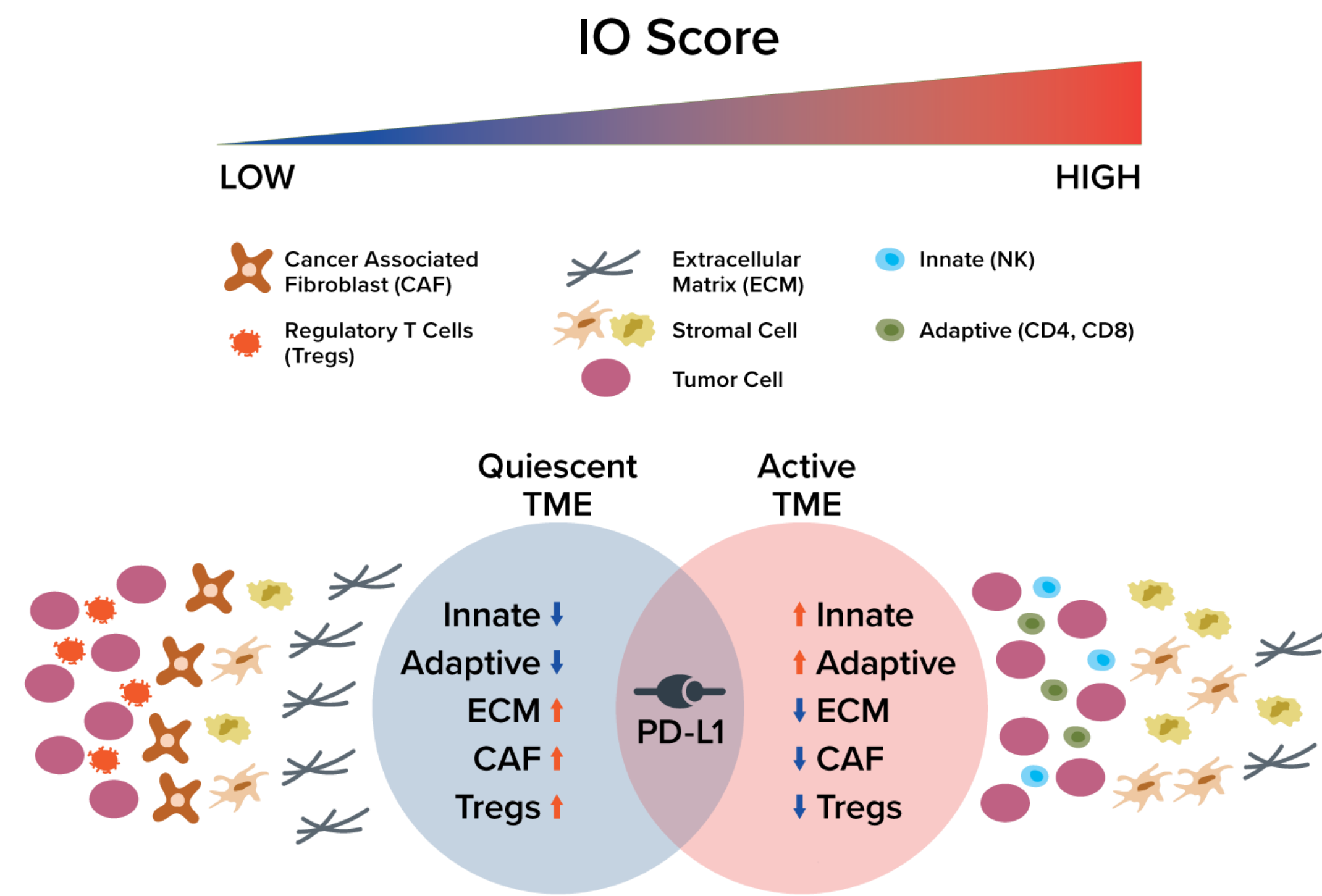

Methods

- The IO score was calculated using the 27-genes as previously described and defined as IO+/IO- based on an existing established threshold.²
- TMB high was determined as ≥ 10 mutations/MB and a composite biomarker of IO+ or TMB high was calculated (IO/TMB).
- Survival was estimated using Kaplan-Meier (K-M) plots and Cox proportional hazard ratios (HRs) were calculated for IO score, TMB high, and IO/TMB.

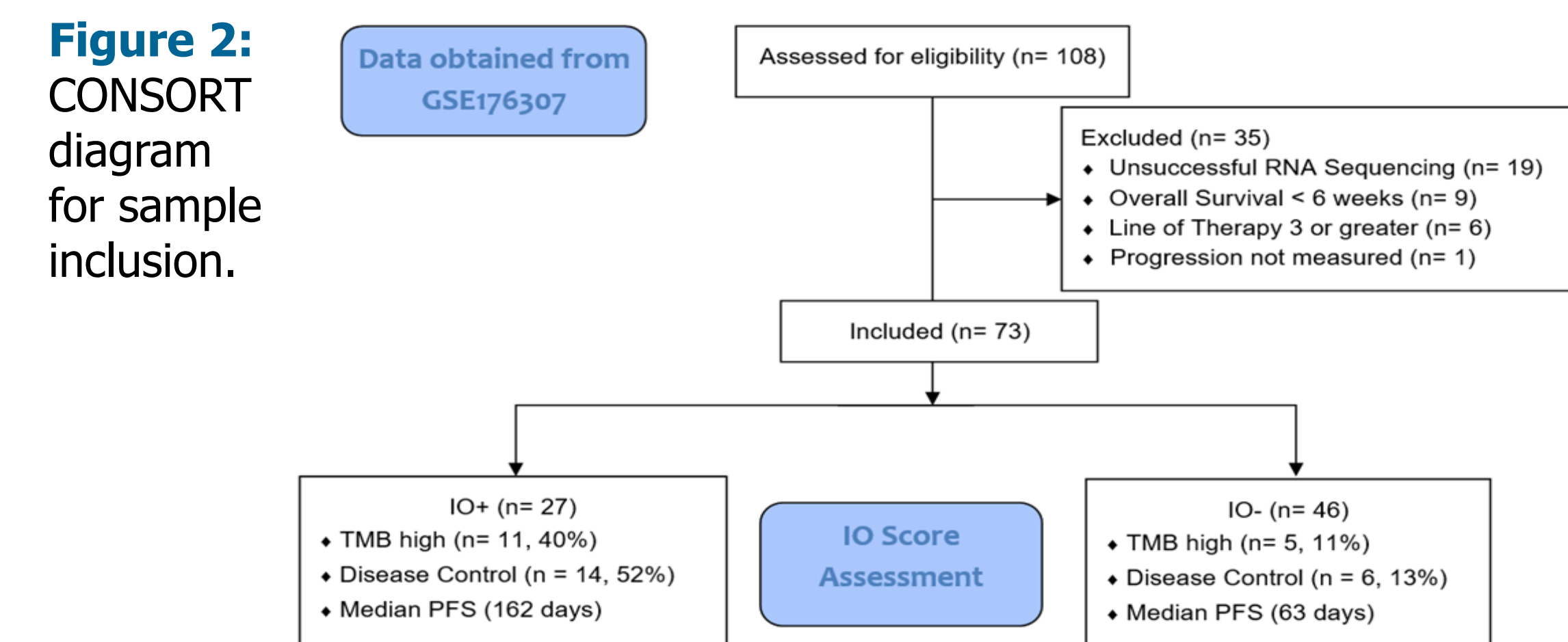

Cohort Demographics

	Cohort (n=73)	IO+ (n=27)	IO- (n=46)	Chi-Square IO+/-
Age at IO Therapy (years)				p=0.88
≤50	3 (4%)	1 (4%)	2 (4%)	
51-60	12 (16%)	5 (19%)	7 (15%)	
61-70	24 (33%)	10 (37%)	14 (30%)	
> 70	34 (47%)	11 (41%)	23 (50%)	
Sex				p=0.41
Female	28 (38%)	12 (44%)	16 (35%)	
Male	45 (62%)	15 (56%)	30 (65%)	
Race				p=0.53
African American	18 (25%)	5 (19%)	13 (28%)	
Caucasian	51 (70%)	21 (78%)	30 (65%)	
Other	4 (5%)	1 (4%)	3 (7%)	
Disease Stage at Dx				p=0.70
Stage 1	6 (8%)	3 (11%)	3 (7%)	
Stage 2	47 (64%)	19 (70%)	28 (61%)	
Stage 3	15 (21%)	4 (15%)	11 (24%)	
Stage 4	4 (5%)	1 (4%)	3 (7%)	
Unavailable	1 (1%)	0	1 (2%)	
Histology				p=0.38
TCC	51 (70%)	20 (74%)	31 (67%)	
TCC w/ Squamous	10 (14%)	5 (19%)	5 (11%)	
TCC with Other non-TCC	11 (15%)	2 (7%)	9 (20%)	
non-TCC	1 (1%)	0	1 (2%)	
FGFR3 Status				p=0.61
Mutated	14 (19%)	6 (22%)	8 (17%)	
Wild-Type	59 (81%)	21 (78%)	38 (83%)	
ECOG Performance Status				p=0.23
0	18 (25%)	5 (19%)	13 (28%)	
1	28 (38%)	9 (33%)	19 (41%)	
2	11 (15%)	4 (15%)	7 (15%)	
3	2 (3%)	2 (7%)	0	
Unavailable	14 (19%)	7 (26%)	7 (15%)	
ICI Therapy Received				p=0.15
Atezolizumab	26 (36%)	9 (33%)	17 (37%)	
Durvalumab	2 (3%)	0	2 (4%)	
Nivolumab	5 (7%)	0	5 (11%)	
Pembrolizumab	40 (55%)	18 (67%)	22 (48%)	

Results

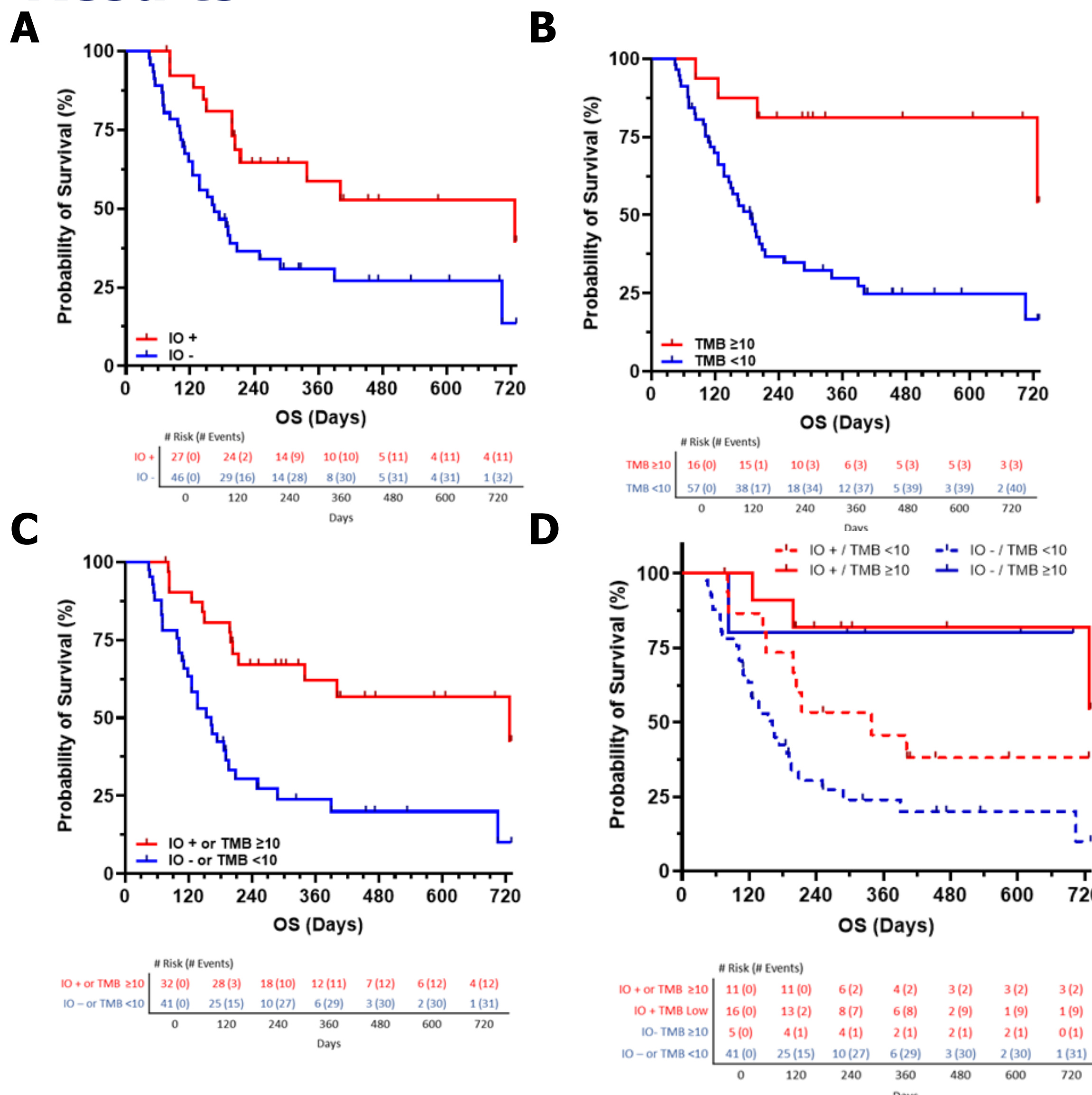


Figure 3: A-C) K-M curve of IO score, TMB, and IO/TMB for up to 2-year overall survival, respectively D) K-M curve of IO score and TMB in bivariate equation.

Results

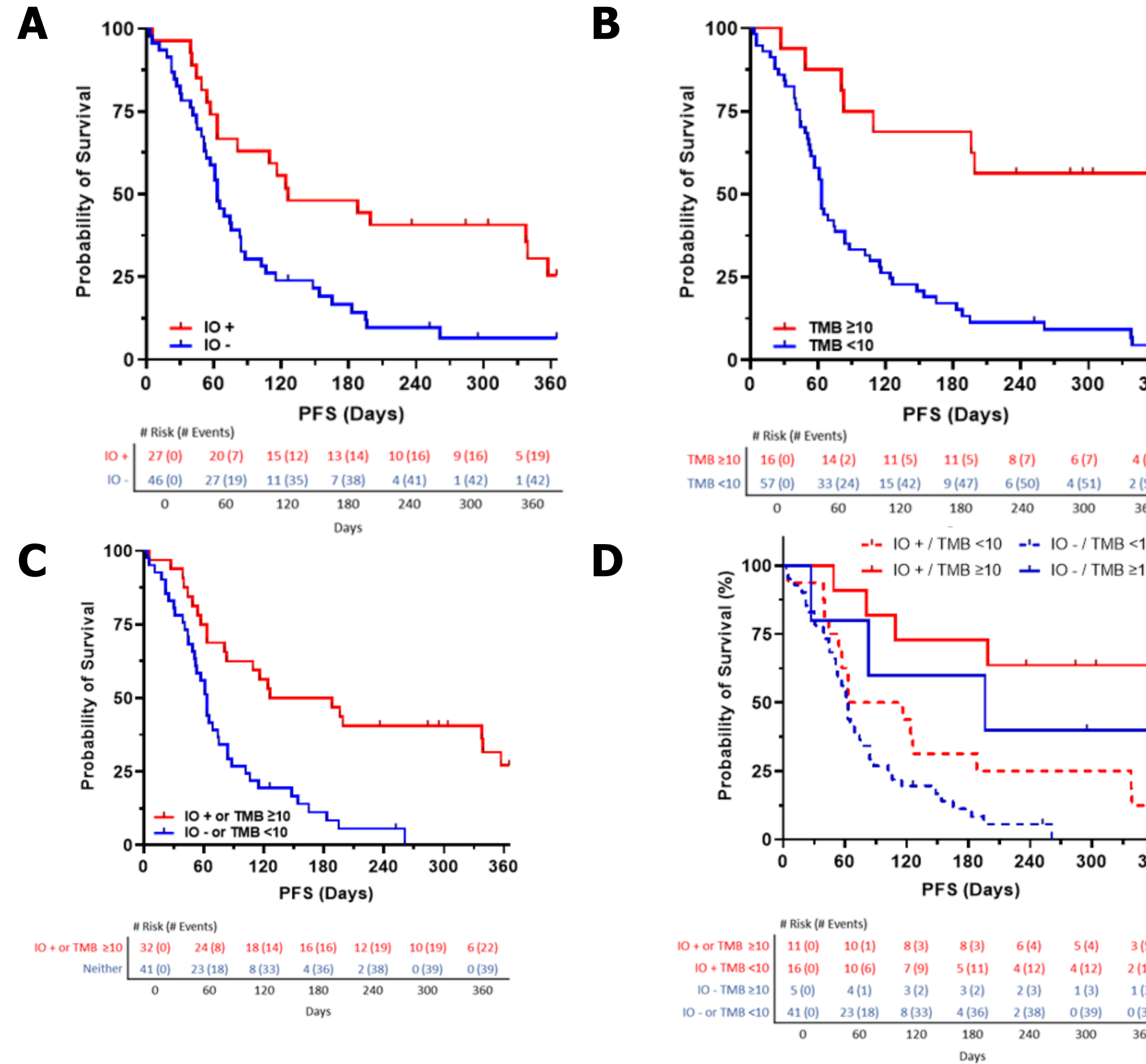


Figure 4: A-C) K-M curve of IO score, TMB, and IO/TMB for up to 1-year PFS, respectively D) K-M curve of IO score and TMB in bivariate equation.

Measure	IO Score	TMB (10 mut/MB)	IO Score or TMB
Biomarker Positive	27 (37%)	16 (22%)	32 (44%)
2-year OS	HR = 0.40; p=0.007	HR = 0.22; p=0.004	HR = 0.31; p=0.0005
1-year PFS	HR = 0.41; p=0.003	HR = 0.32; p=0.0001	HR = 0.32; p=0.0001
Disease Control Odds Ratio	7.18; p=0.007	11.73; p=0.0002	14.36; p=0.0001
PPV / NPV	52% / 87%	69% / 84%	53% / 93%

Table 2: Summary of therapeutic outcome for each of the three biomarkers tested. All three markers were independent of choice of ICI therapy.

Gene	IO+ (n=176)	IO- (n=257)	p=
NECTIN4	57.0	76.4	0.000023
FGFR3	49.3	59.7	0.0068
FGFR1	3.6	4.7	0.056

Table 3: Distribution of *FGFR3*, *FGFR1*, and *NECTIN4* upper quartile from TCGA-BLCA FPKM data. Higher *FGFR3* and *NECTIN4* expression is strongly and *FGFR1* is weakly associated with an IO- TIME.

Conclusion

IO score has demonstrated ability to predict response to ICI therapy in mUC from both the IMvigor210 clinical trial and now confirmed in this real-world cohort.⁴

- IO score is independent and incremental to TMB in predicting response to ICI. Thus, an IO/TMB model would be a rational biomarker.
- Considering the reported 40% ORR with erdafitinib¹ and observed 93% NPV for IO/TMB in ICI therapy; a rational therapeutic paradigm for IO/TMB negative and FGFR altered patients is presented.

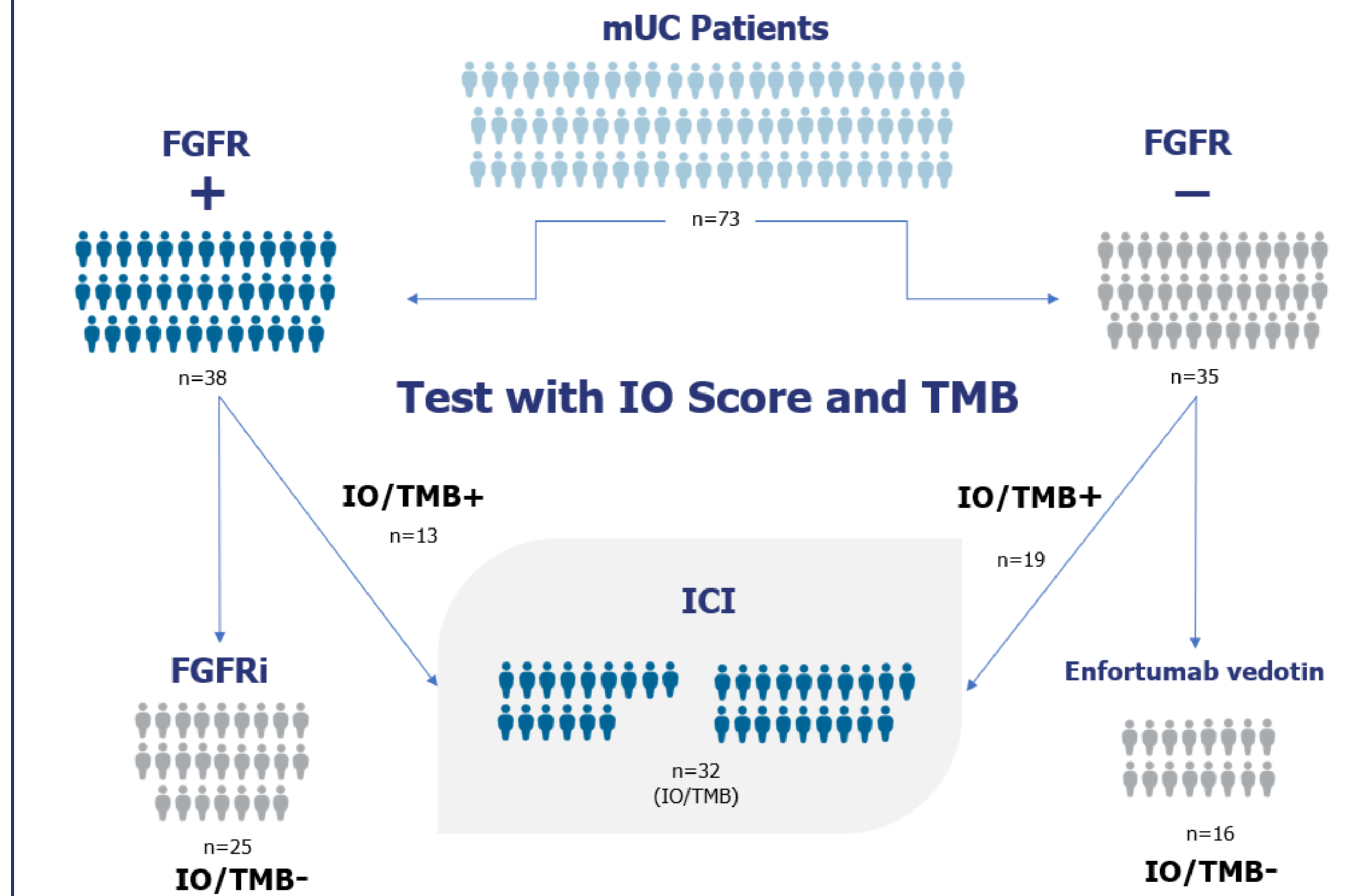


Figure 6: Proposed therapeutic management of platinum ineligible mUC patients.

References

- Rose, T.L.; et. al. Fibroblast growth factor receptor 3 alterations and response to immune checkpoint inhibition in metastatic urothelial cancer: a real-world experience. *B.C.* 2021.
- Nielsen, T.J.; et. al. A novel immuno-oncology algorithm measuring tumor microenvironment to predict response to immunotherapies. *Heliyon.* 2021.
- Iwase, T.; et. al. A Novel Immunomodulatory 27-Gene Signature to Predict Response to Neoadjuvant Immunochemotherapy for Primary Triple-Negative Breast Cancer. *Cancers.* 2021.
- Seitz, R.; et. al. Validation of a 27-gene immuno-oncology algorithm in metastatic urothelial carcinoma treated with an immune checkpoint inhibitor. *ASCO.* 2021.

